

Instant Hot Packs

Procure Instant Hot Packs helps to quickly alleviate inflammation, pain and swelling for fast, soothing relief. Immediate hot therapy relief. Simply squeeze and knead to activate. Once activated, these packs will reach a temperature of 110°F/43°C. Apply hot pack to desired area. Dispose of the hot pack after use, single use only.



Benefits:

- Instant relief minor injuries
- For minor sprains & strains
- Disposable, single use only
- 6x9 inches

Instructions for Use:



Grasp outer edges of pack at the arrows above, SQUEEZE firmly to activate.



KNEAD pack well to mix contents after activation.



APPLY Hot Pack to desired area, Approximate peak temperature 110°F/43°C.



DISPOSE of used pack in proper receptacle.

Visit Us Online at:
www.ProCureProducts.com

Product	Size	Reorder #	Units
Hot Packs	6" x 9"	PC1301	24 Hot Packs/Case