

## Use and Disposal of single-patient Blood Glucose Monitoring Kit for C-diff patient/resident


- *The assigned blood glucose meter will be left in the patient's room and used only for him/her during the time they have C-diff.*
- *Do NOT remove this device from the C-diff resident's room*
- *Once the patient/resident has been cleared from C-diff isolation, you must dispose of the blood glucose meter.*

### Materials Needed:

- Procure Meter that is specific for C-diff patient/resident
- Gloves
- Disinfecting wipe (Super Sani Cloth purple top)



Always wash hands and wear gloves when performing blood glucose testing, and when cleaning and disinfecting the meter; contact with blood presents a potential infection risk.

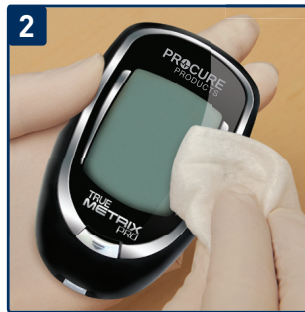

**To Clean:** Make sure meter is off and a test strip is not inserted.\*




Using ONLY Super Sani-Cloth wipes, rub the entire outside of the meter using 3 circular wiping motions with moderate pressure on the front, back, left side, right side, top and bottom of the meter.



Repeat as needed until all surfaces are visibly clean. Discard used wipes.



**To Disinfect:** Using fresh wipes, ensure that the outside surfaces of the meter stays wet for 2 minutes. NEVER put meter in liquids or allow any liquids to enter the test port.\*

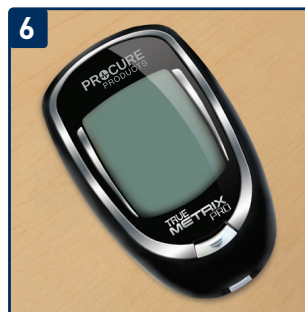

Let meter thoroughly air dry before using it to test.



Properly dispose of gloves and wipes after cleaning. Wash hands after removing gloves.

\* For any further questions, please refer to the cleaning and disinfection procedures as described in the TRUE METRIX PRO Blood Glucose System Owner's Manual.