


Cleaning and Disinfecting

TRUE METRIX® PRO Professional Monitoring Blood Glucose Meters*

A GUIDE FOR MULTIPLE-PATIENT USE FACILITIES


For quick reference only, not intended as a substitute for complete instructions. Please read the entire Owner's Booklet and product instructions before cleaning and disinfecting for the first time and before using this guide. All meters that are shared between patients must be cleaned and disinfected after use with each patient to prevent the transmission of bloodborne pathogens. This recommendation complies with the updated CMS, CDC, and FDA best practices guidelines for assisted blood glucose monitoring. (See backside.)

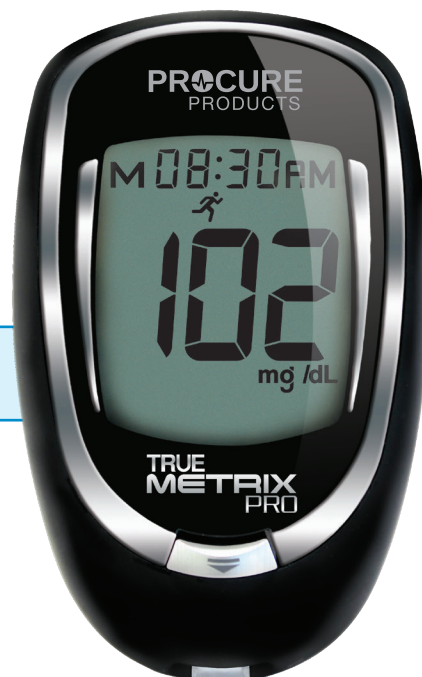

To clean and disinfect the meter, use ONLY PDI Super Sani-Cloth® Germicidal Disposable Wipes or any disinfectant product with the EPA reg. no. 9480-4.¹



Always wash hands and wear gloves when performing blood glucose testing, and when cleaning and disinfecting the meter; contact with blood presents a potential infection risk.

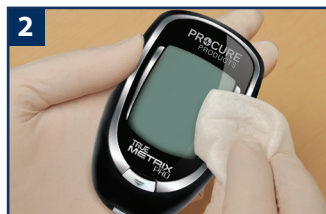

To Clean: Make sure meter is off and a test strip is not inserted.




Using ONLY Super Sani-Cloth wipes, rub the entire outside of the meter using 3 circular wiping motions with moderate pressure on the front, back, left side, right side, top and bottom of the meter.



Repeat as needed until all surfaces are visibly clean. Discard used wipes.



To Disinfect: Using fresh wipes, ensure that the outside surfaces of the meter stays wet for 2 minutes. NEVER put meter in liquids or allow any liquids to enter the test port.

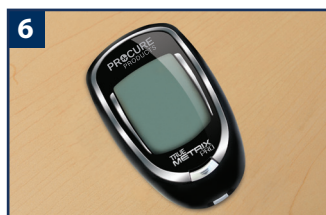

Let meter thoroughly air dry before using it to test.



Properly dispose of gloves and wipes after cleaning. Wash hands after removing gloves.

Important Health And Safety Information

Healthcare professionals should adhere to standard precautions and disinfection procedures when handling or using this device for testing. All parts of the TRUE METRIX® PRO Professional Monitoring Blood Glucose System are considered potentially infectious and capable of transmitting bloodborne pathogens.^{2,3} For more information about standard precautions and practices, please visit <http://www.cdc.gov/hicpac/2007IP/2007isolationPrecautions.html>.



Please Note:

After Cleaning and Disinfecting, verify that the system is working properly by performing an Automatic Self-Test (see *Getting Started* section of the Owner's Booklet).

Do not use the meter and call Customer Care for assistance toll free at 1-800-803-6025 Monday - Friday, 8AM-8PM EST, if:

- Meter Display appears cloudy or any display segments are missing
- Markings on meter, including back meter label, are coming off or missing
- ".", "▶" or "◀" Buttons are hard to push on meter or do not work
- Unable to insert test strip into Test Port
- Automatic Self-Test gives an error message

Note: Other disinfectants have not been tested. The effect of other disinfectants used interchangeably has not been tested with the meter. Use of disinfectants other than Super Sani-Cloth Wipes may damage meter.

Note: Super Sani-Cloth Wipes have been tested on the meter for a total of 10,950 cleaning and disinfecting cycles, which is equal to cleaning and disinfecting the meter 10 times per day for a 3 year period. The use life of the meter is 3 years.

References:

1. Super Sani-Cloth wipes are commercially available from a variety of distributors or visit www.pdipdi.com. For more details about cleaning and disinfecting, please contact our Customer Care department at 1.800.803.6025 Monday - Friday, 8AM-8PM EST.
2. FDA Communication: Use of Fingerstick Devices on More Than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication. Available at <https://wayback.archiveit.org/7993/20170111013014/http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>.
3. Infection Prevention during Blood Glucose Monitoring and Insulin Administration. Available at <http://www.cdc.gov/injectionsafety/blood-glucose-monitoring.html>

NICO-3144PCR 11/25 © 2025 Trividia Health, Inc. TRUE METRIX and the Trividia Health logo are trademarks of Trividia Health, Inc.

All other product names and brand names are trademarks of their respective owners.

* TRUE METRIX is intended for self-monitoring blood glucose only and not for multiple patient use. Only TRUE METRIX PRO is intended for multiple patient use.